

# Kernel Dicom Studio

## Installation/Uninstallation Guide

### Install Kernel Dicom Studio

The installation process of the software involves a few easy steps. Before proceeding, make sure you have the setup file. If you don't have the setup, then go to the website and install it.

Product Page URL : <https://www.nucleustechnologies.com/dicom/viewer/>

Once you get the installer file, follow the steps below to install the software accurately:

- ✓ Locate the **Kernel Dicom Studio installer** file and double click on it.
- ✓ Next, follow the on-screen instructions.
- ✓ Once installation is done, a success message saying **Setup Installation Complete** appears on screen.
- ✓ Click on the **Finish** button to launch the software.

After successfully completing the installation process, a software shortcut will be added to the **Windows Start Program** menu. You can either launch the software from the Program Menu or create a desktop icon to directly launch it.

## Uninstall Kernel Dicom Studio

You can uninstall Kernel Dicom Studio from your system with two methods listed below:

- ✓ From the Windows **Start** menu
- ✓ From the **Control Panel**

## Uninstall from the Windows Start Menu

- ✓ Click on the **Start** menu and go to **All Programs**.
- ✓ Click on **Kernel Dicom Studio » Uninstall Kernel Dicom Studio**.
- ✓ You will get a warning message before uninstalling the software on the screen.
- ✓ Click on **Yes** to uninstall the software.

## Uninstall from the Control Panel

- ✓ Click on the **Start** button, then **Control Panel**.
- ✓ Go to **Programs and Features** and look for **Kernel Dicom Studio**.
- ✓ Right-click on Kernel Dicom Studio and click on the **Uninstall** button.
- ✓ You will get a warning message before uninstalling the software on the screen.
- ✓ Click on **Yes** to uninstall the software.

Kernel Dicom Studio will be successfully uninstalled from your computer system.